



THE NEULASTA® ONPRO® KIT

An Overview on How To Apply Neulasta® Onpro®




This resource is intended to provide you with an overview and basic information about the Neulasta® Onpro® kit and does not replace the Instructions for Use. Please review the HCP Instructions for Use prior to filling and applying the on-body injector.

 **Neulasta®**
(pegfilgrastim) injection


Get to know the Neulasta® Onpro® kit

A Guide to the Components:

On-body Injector for Neulasta®

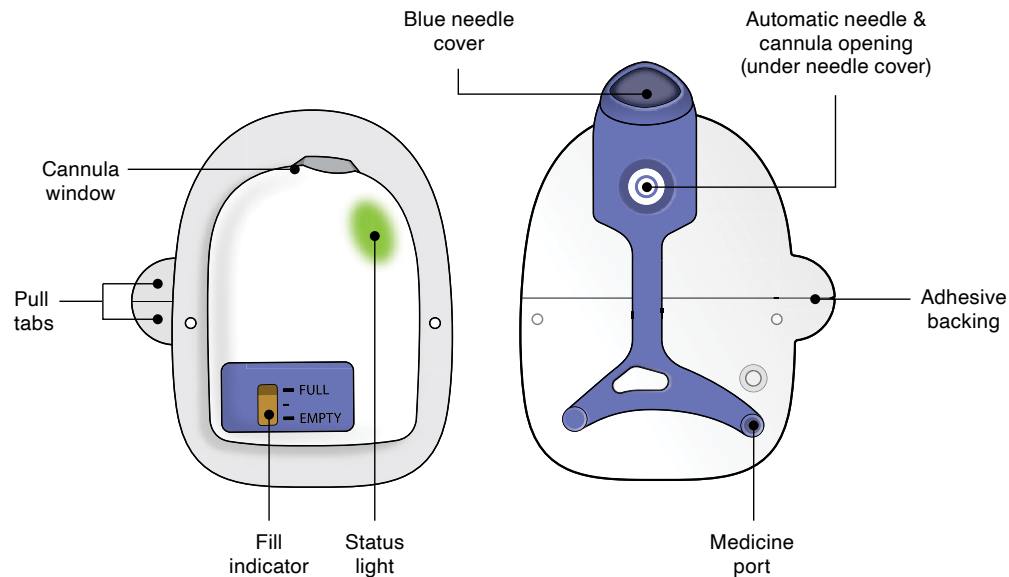

Neulasta® Prefilled Syringe

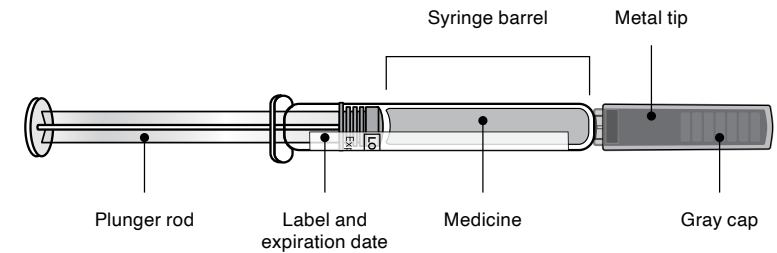

Please note that this flipchart is meant to help you understand how to fill and apply the Neulasta® Onpro® on-body injector—however, it is not meant to replace the Healthcare Provider Instructions for Use (HCP IFU) included with each Neulasta® Onpro® kit. Always consult the HCP IFU. Visit www.NeulastaHCP.com for the HCP IFU.

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES

PREPARE: Ready the kit components



PLACE the syringe tray and the on-body injector tray on a clean, well-lit work surface.



GATHER an alcohol swab, which you will need for applying the on-body injector.



ALLOW the syringe and on-body injector to come naturally to room temperature for 30 minutes prior to activating.



DO NOT warm the kit components using a heat source.

Please note that this flipchart is meant to help you understand how to fill and apply the Neulasta® Onpro® on-body injector—however, it is not meant to replace the Healthcare Provider Instructions for Use (HCP IFU) included with each Neulasta® Onpro® kit. Always consult the HCP IFU. Visit www.NeulastaHCP.com for the HCP IFU.

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH


RESOURCES

PREPARE: Choose the patient's injection site

Choose the Injection Site:

Ask the patient about their ability to monitor and remove the on-body injector.


Patient monitoring

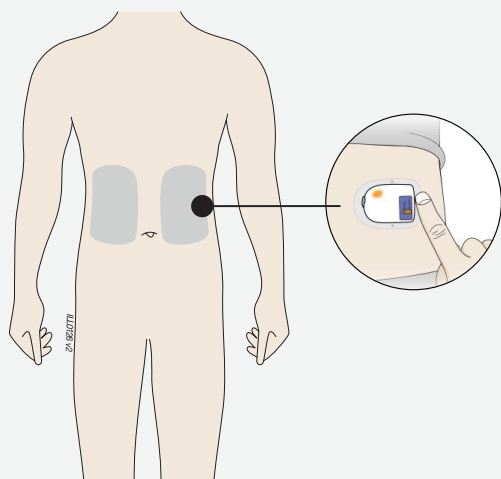

Orient the on-body injector as shown in the picture.

Abdomen:
Use the left or right side of the abdomen, except for a two-inch area right around belly button.

Caregiver monitoring

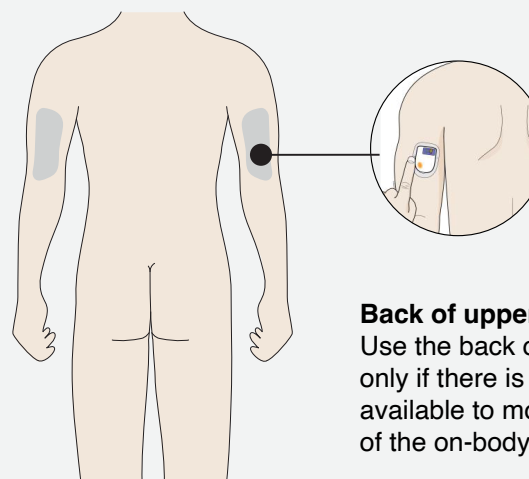

Orient the on-body injector as shown in the picture.

Back of upper arm (triceps):
Use the back of upper arm only if there is a caregiver available to monitor the status of the on-body injector.

- Apply the on-body injector to intact, non-irritated skin

! **DO NOT** apply the on-body injector on surgical sites or areas with scar tissue, moles, or excessive hair. In case of excessive hair, carefully trim hair to get the on-body injector close to the skin.

! **DO NOT** apply the on-body injector on areas where belts, waistbands, or tight clothing may rub against, disturb, or dislodge the on-body injector.

! **DO NOT** apply the on-body injector on areas where the on-body injector will be affected by folds in the skin.

GETTING
TO KNOW

PREPARE

FILL


APPLY

FINISH

RESOURCES


PREPARE: Get ready to fill

CLEAN an area on the injection site larger than the on-body injector adhesive backing.

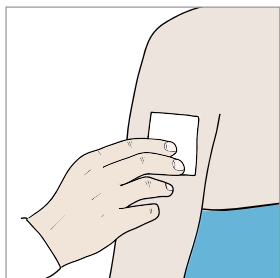


- **Only use alcohol** to clean the application site. This site must be oil-free before applying the device



Allow the skin or skin pad to **completely dry** before attaching the device.

 **DO NOT** touch this area again before attaching the device.

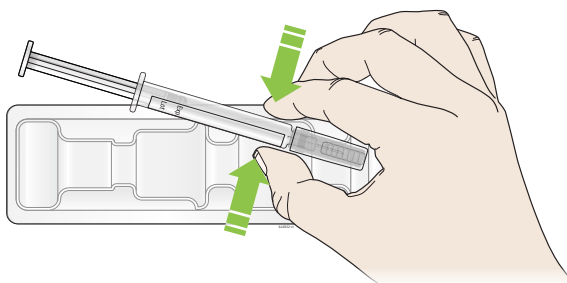
REMOVE the Neulasta[®] prefilled syringe from tray.




For safety reasons:

-  **DO NOT** grasp the gray needle cap.
-  **DO NOT** grasp the plunger rod.

INSPECT Neulasta[®] prefilled syringe.

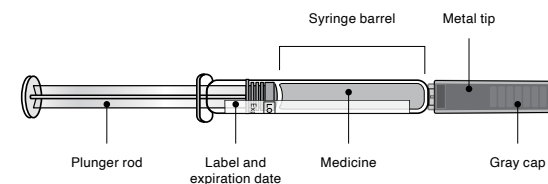

 Neulasta[®] liquid should always be clear and colorless.

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES


FILL: Remove air bubbles in the prefilled syringe

Steps to remove air bubbles:

- 1 **REMOVE** the gray needle cap.
- 2 **GENTLY TAP** the syringe with your finger until air bubbles rise to the top.
- 3 **SLOWLY PUSH** air out of the syringe, taking care to not expel medicine.
- 4 A small droplet at the tip of the needle during air purging is normal.

 **DO NOT** recap the syringe.

Injecting air bubbles could interfere with proper operation of the on-body injector.

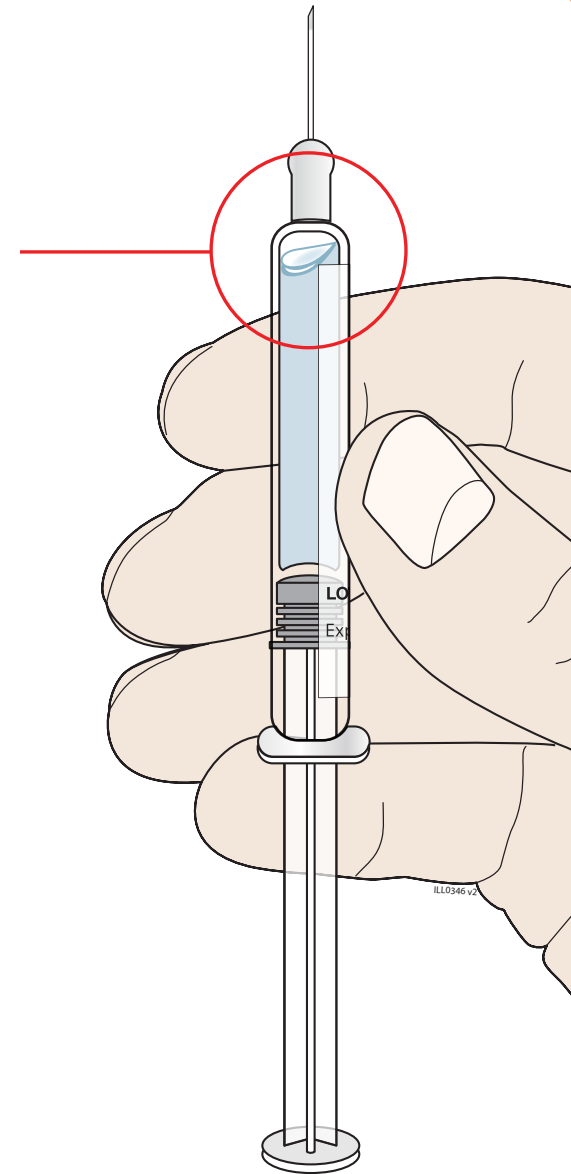

GETTING
TO KNOW

PREPARE

FILL

APPLY




FINISH


RESOURCES

FILL: Insert needle into port

CENTER the needle directly over the medicine port and insert all the way into the port, avoiding sides.

INSERT needle into medicine port at a 90° angle only.

-  **DO NOT** remove the blue needle cover before filling the on-body injector.
-  **DO NOT** insert the needle more than once.
-  **DO NOT** bend the needle.

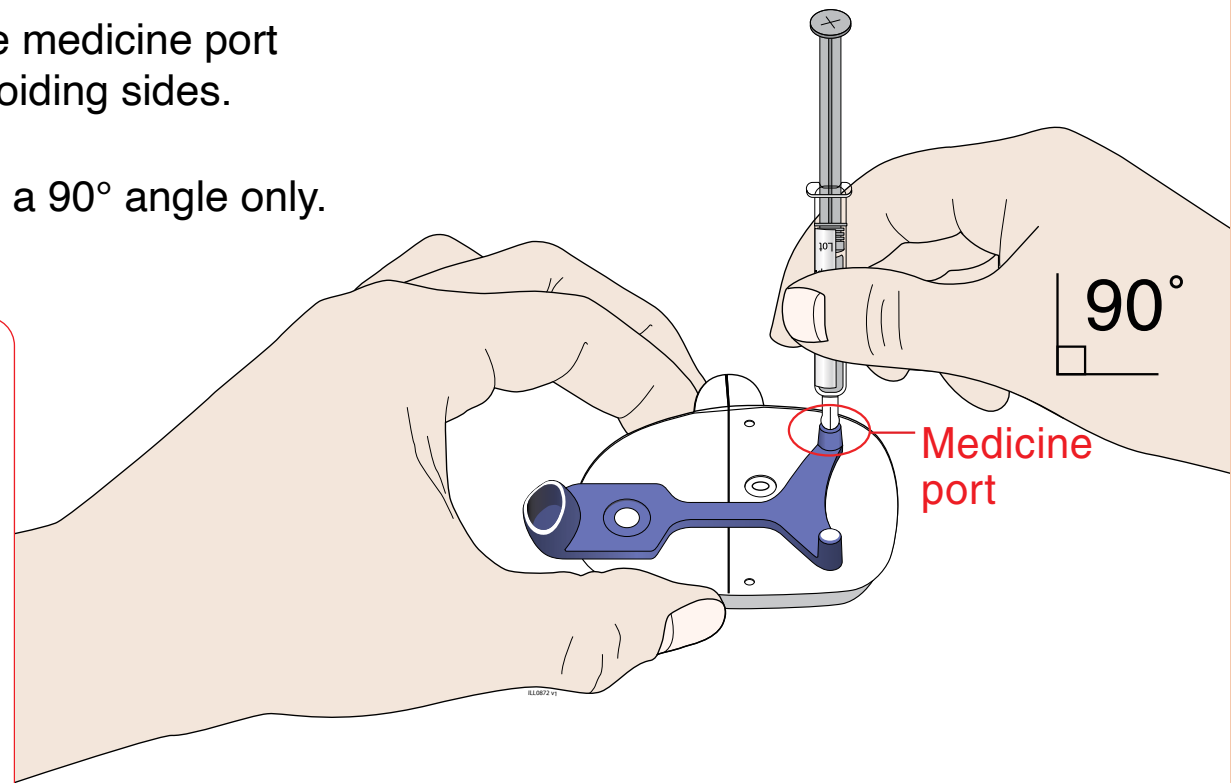

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES

FILL: Empty syringe contents into on-body injector


- 1** **PUSH** the plunger rod to empty entire syringe contents into the on-body injector.

During filling, you will hear beeping.



The status light will flash amber.

Activation light

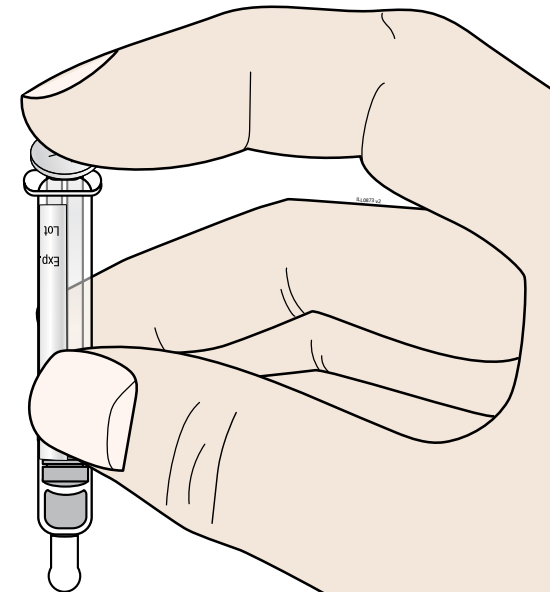

- 2** You now have 3 minutes to apply the device to the skin.



GETTING TO KNOW

PREPARE

FILL

APPLY


FINISH

RESOURCES


FILL: Confirm & prepare to apply on-body injector

- 1 CONFIRM** if the on-body injector is full and the amber light is flashing.

You should see a black line next to FULL on the fill indicator. If this is not the case, do not use. Start again with a new Neulasta® Onpro® kit.

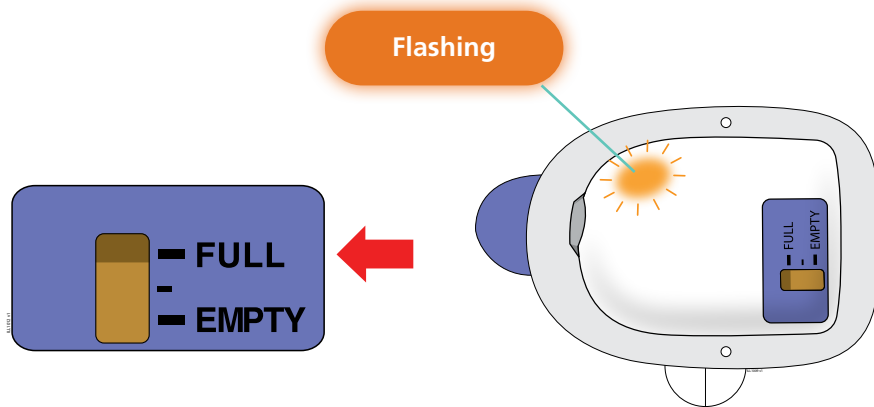

- 2 FIRMLY LIFT AND REMOVE** the blue needle cover away from the on-body injector.

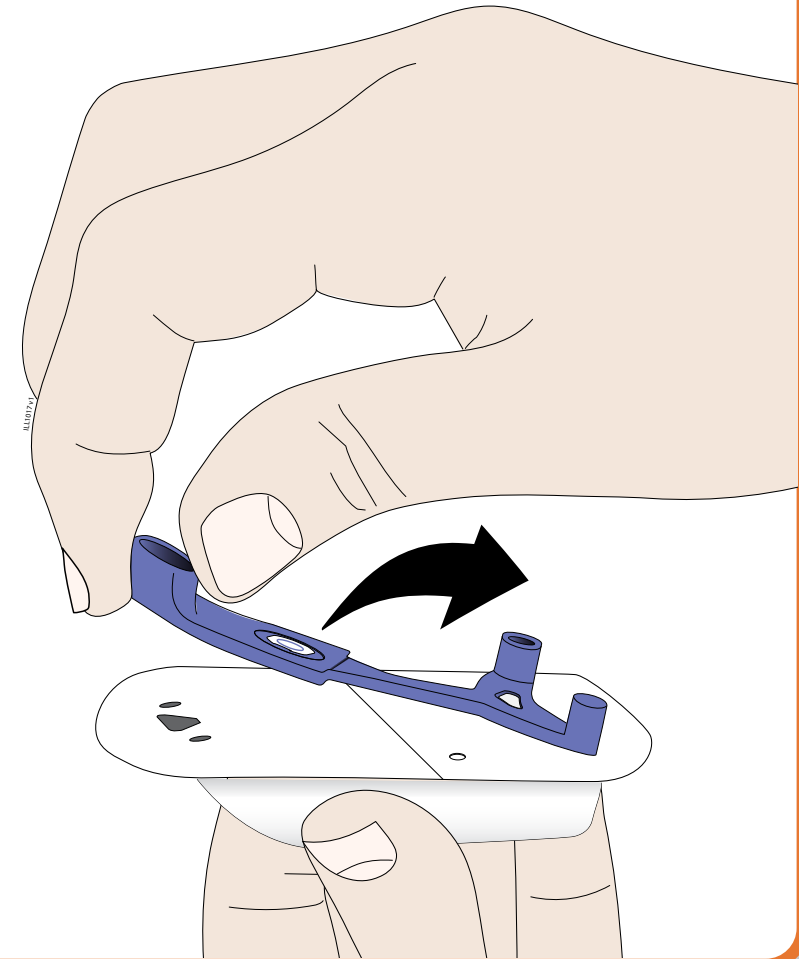

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH


RESOURCES

APPLY: Prepare on-body injector for application

IMPORTANT: Make sure skin is dry prior to applying the on-body injector

PEEL away both pull tabs to show the adhesive.

- !** **DO NOT** touch hands or gloves to the adhesive.
- !** **DO NOT** pull off the adhesive pad or fold it.
- !** **DO NOT** touch or contaminate the automatic needle area.
- !** **DO NOT** use if the needle or cannula is extended past the adhesive or is extended before the on-body injector is placed on the patient.
- !** **DO NOT** place adhesive on skin that is damp.

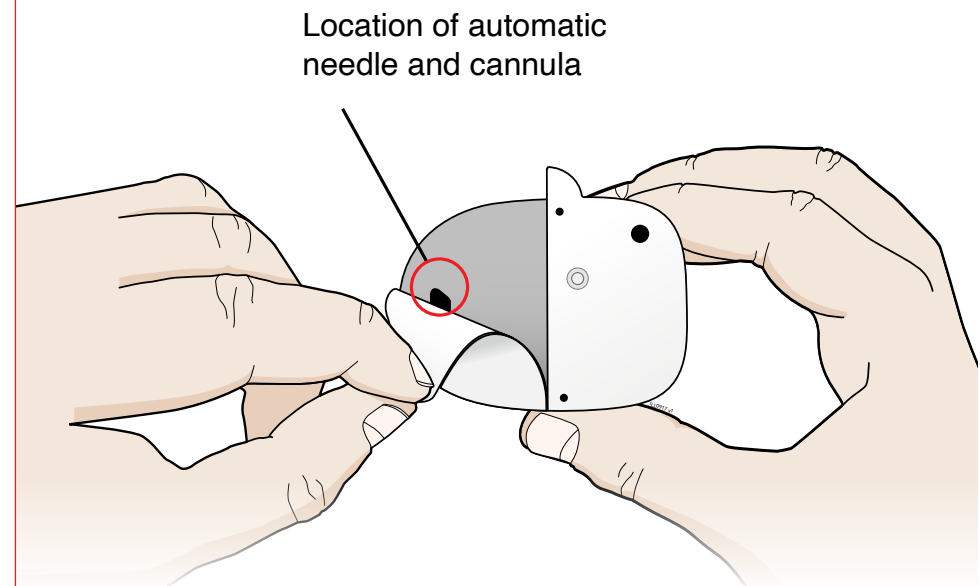

GETTING
TO KNOW

PREPARE



FILL

APPLY

FINISH


RESOURCES

APPLY: Secure the on-body injector to the patient

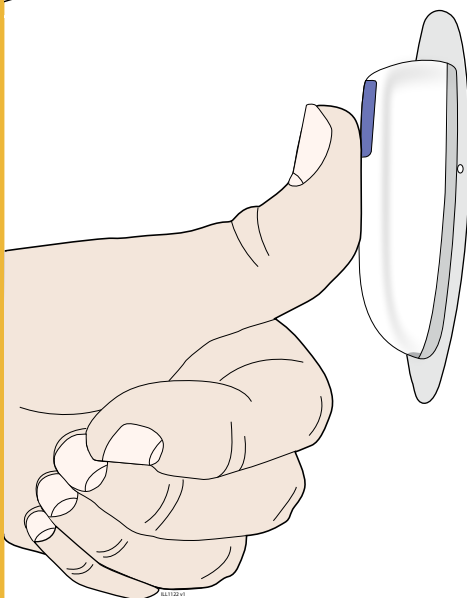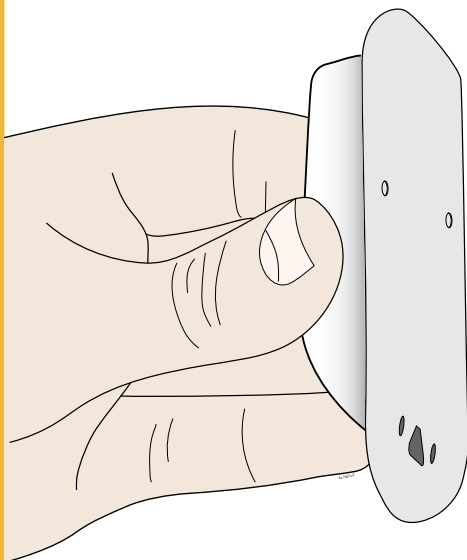

- 1 **GRASP** the on-body injector's plastic case with your fingertips and only by sides, keeping fingers off of the adhesive.

 **DO NOT** touch the adhesive.

 **DO NOT** let the adhesive bend or curl while applying the on-body injector to skin.

- 2 Once on the skin, **PRESS FIRMLY** on the on-body injector to ensure proper adhesion to the patient's skin.
- 3 **PRESS** around the entire adhesive so it lies down without folds or wrinkles.
- 4 **HOLD** the top of the on-body injector and run finger around the adhesive to create a secure attachment.

GETTING
TO KNOW

PREPARE

FILL

APPLY


FINISH

RESOURCES

FINISH: Automatic cannula insertion

After 3 minutes the Status Light turns GREEN, meaning the cannula has been inserted.

The on-body injector alerts will change as follows:

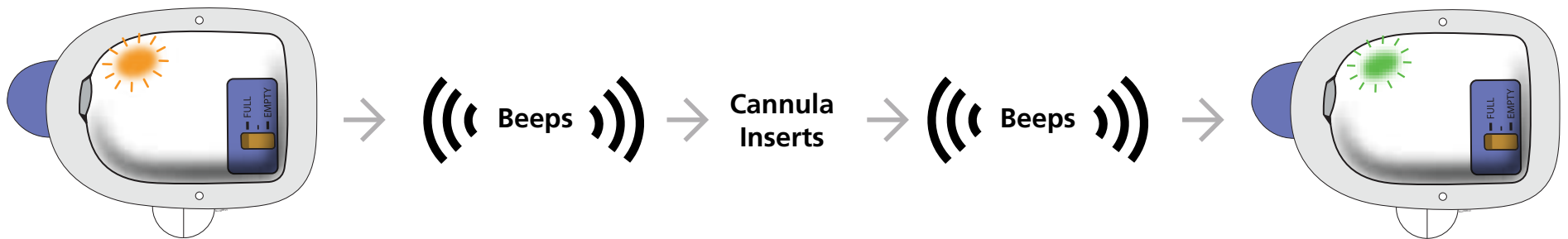

! **DO NOT** remove the on-body injector during cannula insertion to avoid needlestick injury to you or to the patient.

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES

FINISH: Check adhesion quality




Check the quality of adhesion before sending the patient home.

If the adhesive is wrinkled in front of the cannula window or has folds anywhere that prevent the on-body injector from securely adhering:


- 1 REMOVE** the on-body injector.
- 2 START** again with a new kit.
- 3 CALL** Amgen at 1-800-772-6436.



DO NOT remove the on-body injector during cannula insertion to avoid needlestick injury to you or to the patient.

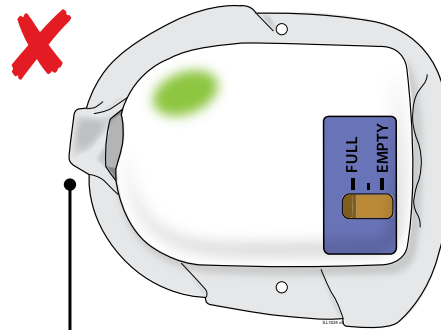

Wrinkled adhesive near cannula window

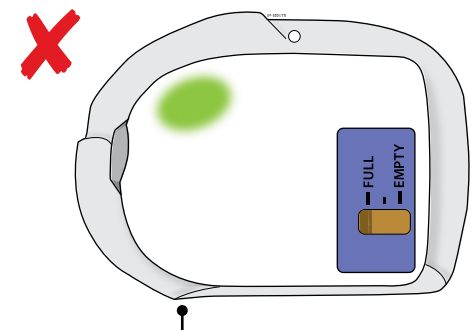

Folded adhesive

GETTING TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES

FINISH: If the patient may benefit from additional adhesion

If you feel additional adhesion is appropriate:

An adhesive extender (called “PodPals™”) that fits around the on-body injector can be obtained at no charge by:



Visiting

www.NeulastaPodPals.com

Scanning this QR Code



Calling

**1-844-MYNEULASTA
(1-844-696-3852)**

(Select option 2, then option 3)



DO NOT use other materials to secure the on-body injector to the patient that could cover audio/visual indicators or compress the on-body injector against the patient’s skin.

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES


FINISH: Prepare the patient

Before sending patients home, make sure they're ready for the next day.

1 FILL IN the dose delivery information on the front of the Patient IFU Booklet.

2 REVIEW the following instructions with your patient:

- The on-body injector will always flash a slow green light to let them know it is working properly
- The patient should keep the on-body injector dry for at least 3 hours after it was placed on their skin
- After approximately 27 hours, the dose delivery will begin. Dose delivery will take about 45 minutes. During this time, the on-body injector will flash a fast GREEN light
- When dose delivery is complete, the on-body injector will sound a long beep, and the status light will turn SOLID GREEN
- Do not remove the on-body injector until the status light is SOLID GREEN
- If the RED error light is flashing, or the adhesive is noticeably wet (saturated), or the on-body injector is dislodged, the patient should contact their healthcare provider immediately as they may need a replacement dose

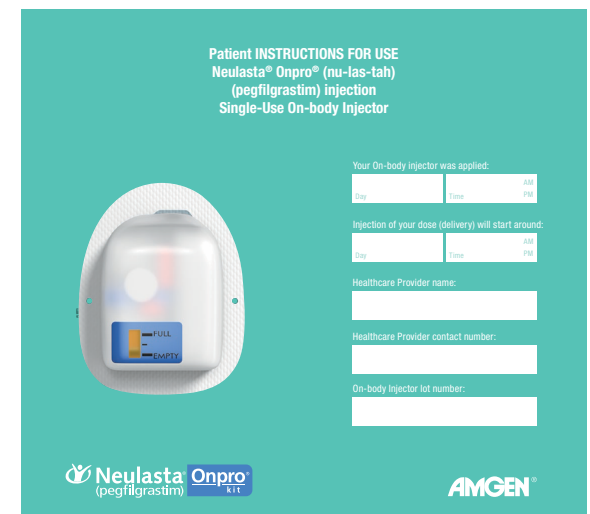

Patient Instructions for Use Booklet

Included with Onpro® kit

GETTING
TO KNOW

PREPARE

FILL

APPLY


FINISH

RESOURCES

Addressing Fall-offs

If at any time your patient reports the device has fallen off:

- They may not have received the full dose
- Patient should contact their healthcare provider immediately as they may need a replacement dose
- Schedule a follow-up appointment with your patient
- Report the incident to Amgen at 1-800-772-6436
- Start over with a new Neulasta® Onpro® kit

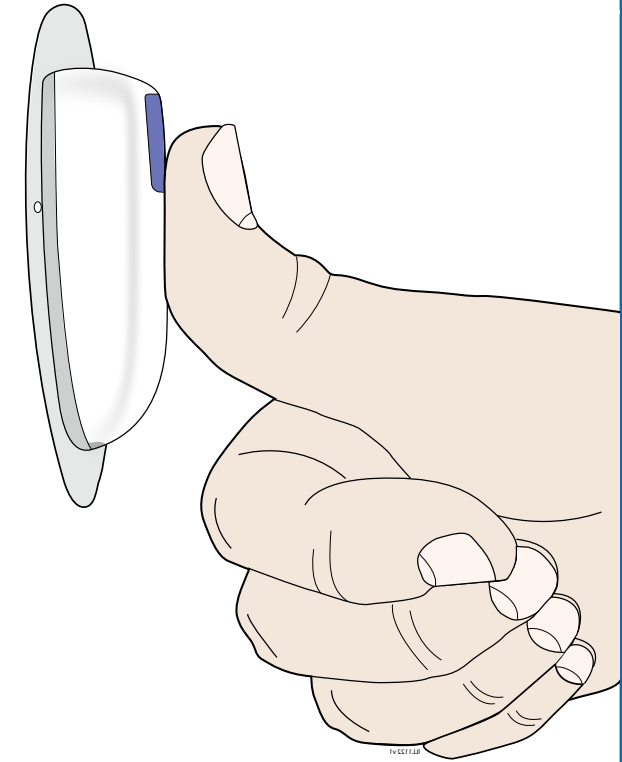

Remind the patient that care should be taken to minimize bumping the device while adhered to the skin.

GETTING
TO KNOW

PREPARE

FILL

APPLY


FINISH

RESOURCES

TROUBLESHOOTING

If you hear beeping, or the status light is flashing **RED**:

- If at any time the on-body injector beeps continuously for 5 minutes, and the status light is flashing **RED**, take the on-body injector off of the patient
- **DO NOT** apply or leave the on-body injector on the patient if **RED** error light is on.
- In all cases, do not use
 - Start over with a new Neulasta® Onpro® kit
 - Call Amgen at 1-800-772-6436

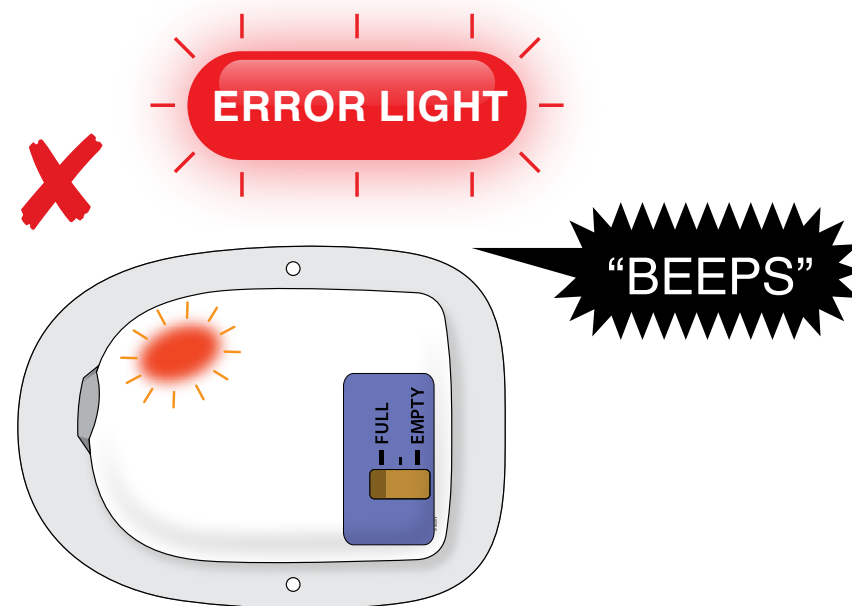

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES



TROUBLESHOOTING

If your patient reports the status light is flashing **RED**:

- They may not have received the full dose
- Schedule a follow-up appointment with your patient
- Report the incident to Amgen at 1-800-772-6436

If your patient reports the adhesive is saturated with fluid or the on-body injector is dripping:

- If the patient reports an on-body injector leak, they may not have received the full dose
- Schedule a follow-up appointment with your patient
- Report the incident to Amgen at 1-800-772-6436

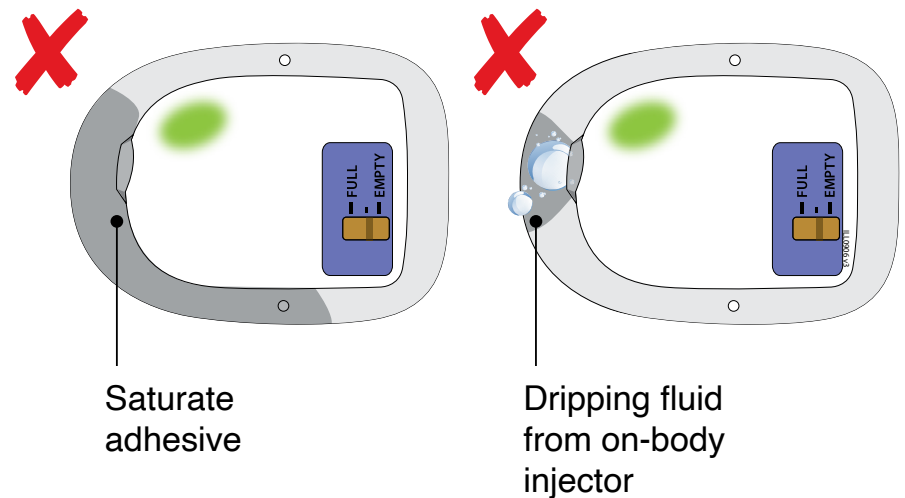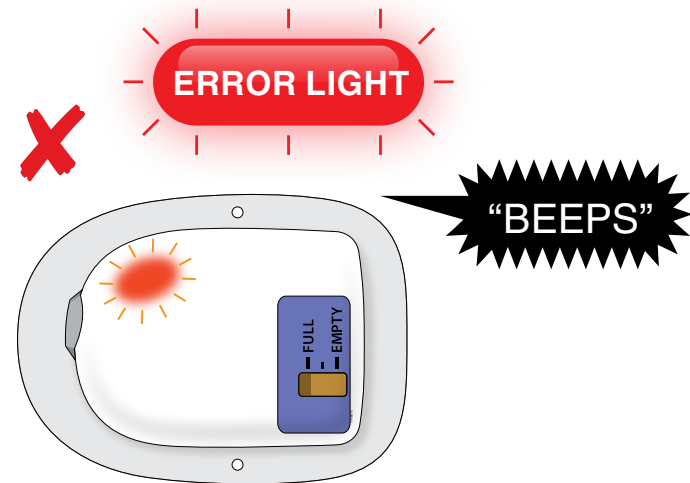

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES

Onpro® kit healthcare provider support is available


Additional educational resources are available to you.

HCP Demonstration Video



See how to apply Neulasta® Onpro®.

Onpro® HCP Learning Center

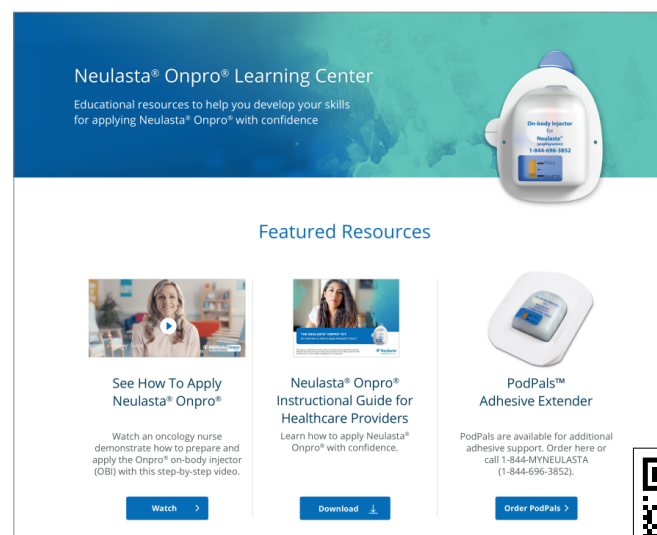

Learn more about additional resources to help you apply Neulasta® Onpro® at www.NeulastaHCP.com.

GETTING TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES

Onpro[®] kit patient support is available

If your patients have any questions along the way, support is just a phone call away.

Live Support

Patients get 24/7 live telephone support for questions about Neulasta[®] Onpro[®]



1-844-MYNEULASTA
(1-844-696-3852)

(Select option 2, then option 3)

Online Resources

Your patients can view instructional videos and access other helpful resources at Neulasta.com



Neulasta.com

GETTING
TO KNOW

PREPARE

FILL

APPLY

FINISH

RESOURCES